

Codonics Horizon Operating Software

Technical Brief

Version 4.2.0 Release Notes

Summary

Horizon® 4.2.0 software provides support for improvements to the manual film calibration and cassette slot selection mechanism.

Products Affected

GS/G2/G1 Imagers containing newer hardware identified by the serial number prefix 150C are affected. Please contact your Codonics representative for further information.

The Imager Software Version can be determined using the Control Panel by viewing the Imager Info screen from the Utilities menu.

NOTE: Horizon Imagers with serial number prefix 150C cannot be downgraded to any previous software version below 4.0.0. Imagers with a serial number prefix other than 150C cannot have their software upgraded to 4.0.0.

Product Improvements (Perfective Updates)

- ◆ Simplified the printed instructions on printed sheets for manual film calibration instructions.
- ◆ Updated the software slot selection mechanism to use the cassette seen by this imager first. The imager will always pick from the “oldest” cassette of the desired type and size of media currently installed in the imager.

- ◆ The FDA UDI GTIN (14-digit number) has been removed from Horizon software. Since Horizon software only works on Horizon imagers a UDI number is not required for software.

Defects Corrected (Corrective Updates)

- ◆ Corrected the film calibration menu to display the value currently set for the specific cassette being calibrated.
- ◆ Changed the error message that occurs when a film calibration print is made but the user does not enter a calibration value. Message made to be more useful.

Known Common Issues

- ◆ Previous issues can be found in previous Release Notes.

Known Uncommon Issues

- ◆ Previous issues can be found in previous Release Notes.

Technical Support

If problems occur during software installation, please contact Codonics Technical Support at any time.

Phone: +1.440.243.1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.
Copyright © 2017 Codonics, Inc. Printed in the U.S.A.
Part No. 901-462-007.01.



17991 Englewood Drive
Middleburg Heights, OH
44130 USA
+1.440.243.1198
+1.440.243.1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Trading Co, Ltd.
317 Xianxia Rd. Building B
Unit 1412
Changning Dist., Shanghai
P.R. China, 200051
86-21-62787701
86-21-62787719 Fax

Codonics Limited KK
AQUACITY 9F,
4-16-23, Shibaura
Minato-ku, Tokyo,
108-0023 JAPAN
81-3-5730-2297
81-3-5730-2295 Fax