

Codonics Infinity C Power Adapter Spare Part

Technical Brief

Overview

This document describes the procedure for replacing the AC power adapter on the Codonics® Infinity™ C.

Remove the Old AC Power Adapter

1. If the Infinity C is running, power-off the system using the front panel power button (see figure 1).



Figure: 1

2. Unplug the AC power line from the power outlet or UPS.
3. Release the retaining clip and disconnect the power cord from the back of the Infinity C chassis (see figure 2).

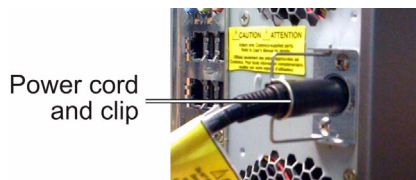


Figure: 2

4. Dispose of the old power supply in accordance with local regulations.

Connect the New AC Power Adapter

1. Remove the new AC power adapter from shipping material.
2. Connect the AC power line to the AC power adapter (see figure 3).

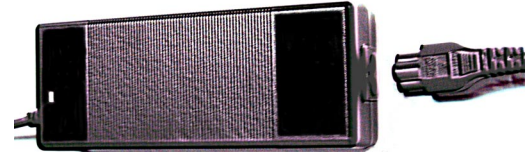


Figure: 3

3. Connect the power cord to the power connector on the back of the chassis and secure with the retaining clip (see figure 2)
4. Plug the AC power line into a power outlet or UPS.
5. Power-on Infinity C using the front panel power button.

Technical Support

If problems occur that are not covered by this Tech Brief, contact Codonics Technical Support between the hours of 8:30AM and 5:30PM EST (weekends and U.S. holidays excluded).

Phone: +1 440-243-1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2008 by Codonics, Inc. Printed in the U.S.A. Part No. 901-201-001 Rev. 01



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295