

Codonics Virtua CD-R Premium Print-Surface Media

CD-R Premium Print-Surface Media

Virtua™ CD-R Premium Print-Surface Media is a high quality recordable disc with a water and smudge-resistant printable surface. It provides a powerful advantage of fast-drying printing that won't wash away or smear. Virtua CD-R Premium Print-Surface Media utilizes a completely sealed inner and outer edge that virtually eliminates oxidation, thus preserving data far longer than budget-priced, lower quality media. Discs are optimized and tested for use with the Codonics Virtua Medical Disc Publisher. Only approved/recommended sources of Virtua CD-R Media should be used with the Virtua Medical Disc Publisher to insure correct data, and labeling integrity.

Storage


Avoid using or storing the discs in an environment with high temperature or humidity. This may cause ink to transfer resulting in smudges or discs sticking to each other.

- ◆ Storage: 10 to 70°C; 3 to 95% RH non-condensing.
- ◆ Recommended Storage: 23°C; 50% RH.
- ◆ Avoid exposure to direct sunlight or strong light.
- ◆ It is recommended that the discs remain sealed in disc holders until used. This will protect the unrecorded discs from damage.



Disc Handling

- ◆ Protect the recording surface of the discs from scratches, fingerprints, dust and smears.
- ◆ Hold the discs along the outer edges or center hole.

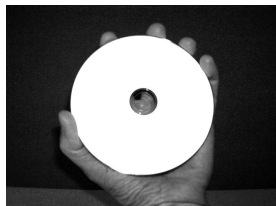

- ◆ Never touch the recording surface, printed label surface, or place it face-down on hard surface.
- ◆ Clean dirty discs with soft, dry cloth. Commercially available disc cleaner or ethyl alcohol may also be used.

CAUTION! Do not use gasoline, kerosene, lacquer-thinner, benzene, anti-static agents or LP record cleaners as it may damage the recording surface of the disc.

Disc Holder

Once opened, protect the discs by storing them in the covered disc holder.

CAUTION! Make sure the clear disc at the top of the roll is removed.



Loading Virtua

CAUTION! Make sure the Codonics Virtua Medical Disc Publisher is not recording/printing a disc.

1. Open the cover to access input bins.



2. Gently loosen the discs prior to placing them in the disc bins. This will help reduce discs sticking together.

- Insert discs in the input bin. Each input bin holds a maximum of 50 discs.



- Close the Cover.

Printing/Usage

- Printed discs should be allowed to dry after printing.
- If labeling manually, write only on the label (white) side using a soft, felt tip marker.

IMPORTANT! Never use a ballpoint pen or hard tipped writing instrument.

Recording/Printing

After recording/labeling, individual discs should be stored in Tyvek Sleeves or jewel cases for best results.

IMPORTANT! Do not place a wet disc in a Tyvek Sleeve. This could damage the recorded data or labeling on the disc.

Cleaning

- For best ongoing performance, clean the Virtua discharge ramp with distilled water using a lint-free cloth once every couple weeks or as needed. Let air dry for five minutes before using Virtua.

Ordering Information

| Catalog Number | Item | Quantity |
|-----------------------|------------------------------------|-----------------|
| 1VCA-RC933 | Codonics Virtua CD-R Premium Media | 300 CDs |
| V-SLEEVES | Codonics Virtua Tyvek Sleeves | 1000 Sleeves |

Technical Support

For questions or problems with Codonics® Premium CD-R Media, please contact Codonics Technical Support. 8:30AM - 5:30PM EST, Monday-Friday (U.S. holidays excluded):

Phone: 440-243-1198

Email: support@codonics.com

Website: www.codonics.com

Get it all with just one call
1-800-444-1198



17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2008 by Codonics, Inc. Printed in the U.S.A. Part No. 901-347-001 Rev. 02.